

AT TABLE SETUP



1. SCOPE

- 1.1. To provide instruction for setting up an AT Table.

2. REQUIRED COMPONENTS

- 2.1. AT Table
- 2.2. 2nd Shelf (optional)
- 2.3. Soft-sided, Travel Case (optional)

3. SETUP

- 3.1. Place the table on its top so the folded legs are facing up. Use a non-marring surface such as grass, turf, carpet, or the soft-sided, travel case so the AT Table top does not get scratched.

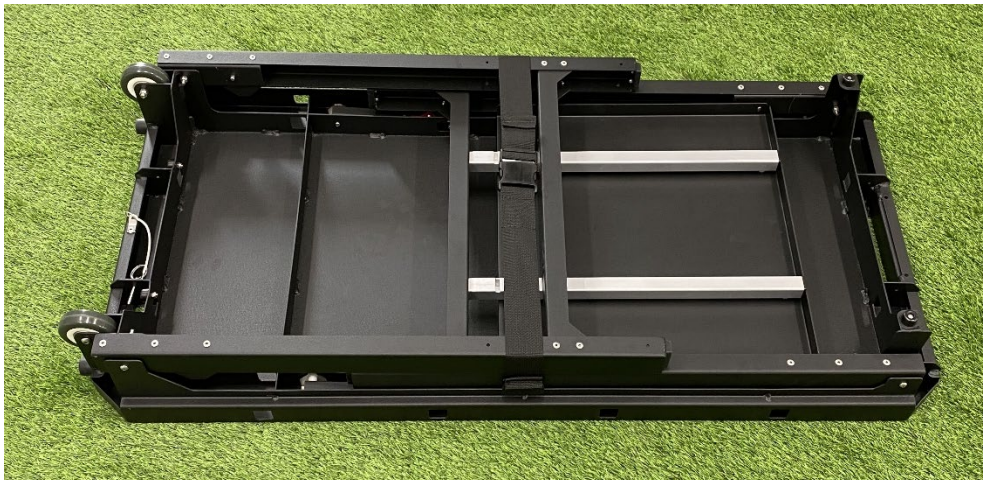


Figure 3.1 Positioning the AT Table

3.2. Unbuckle the retaining strap and move it out of the way.



Figure 3.2 Unbuckle the Retaining Strap

3.3. Unfold the table legs beginning with the top leg and then the bottom leg.



Figure 3.3 Unfold the Legs

- 3.4. Beginning with the left leg set, raise the left pair of trusses and position the end notches beneath the contact surfaces on the left leg. Do not slide the contact surfaces all the way into the truss notch until both trusses have been started.

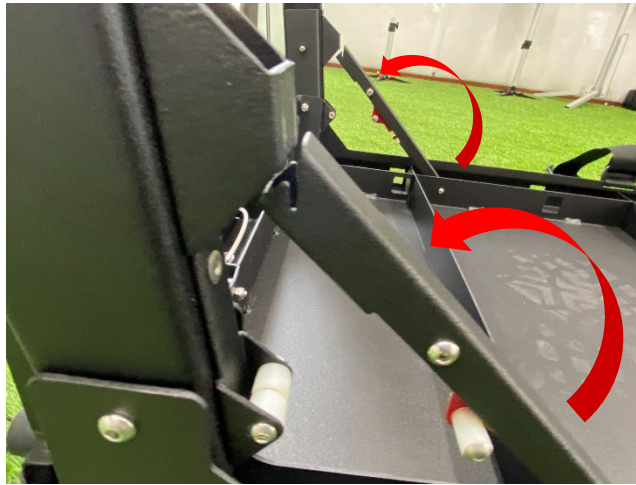


Figure 0 Position the Trusses

- 3.4.1. Pull the horizontal channel connecting the legs towards the middle of the table to seat the contact surfaces of the legs into the truss notches.
- 3.5. Pivot the truss locks into place until they are fully engaged around the stop pins.



Figure 3.5 Position the Truss Locks

- 3.6. Repeat Steps 0-3.5 for the right leg set.
- 3.6.1. **DO NOT USE THE TABLE UNLESS ALL FOUR TRUSS LOCKS ARE FULLY ENGAGED.**
- 3.7. Resecure the retaining strap underneath the table.
- 3.8. Flip the AT Table over so it is standing upright on its legs.

4. SHELF USAGE

- 4.1. Remove the shelf from its stowed position between the AT Table legs by depressing the spring buttons at the top of each side.



Figure 4.1 Removing the Shelf

- 4.2. The shelf can be installed using any two adjacent square cutouts positioned around the table top. The shelf must be inserted at a downward angle in order to pass through both sets of supports. Once the shelf edge makes contact with the AT Table top, rotate the shelf down until it sits flat.



Figure 4.2 Attaching the Shelf

- 4.3. When using the AT Table and shelf in the dolly configuration, the secondary locking pin must be inserted through the AT Table top and shelf tube.

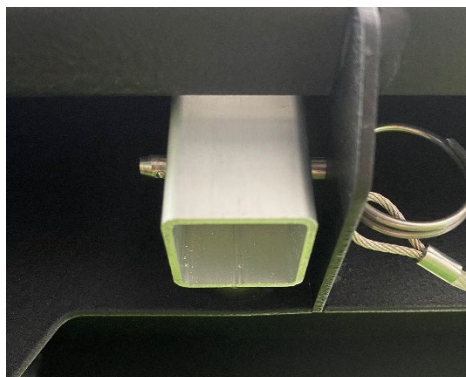


Figure 4.3 Dolly Configuration

CONTROLLED DOCUMENT REVISION HISTORY

Revision: A
Revision Created: 04/05/2022
Approved By: Patrick Powell
Revision Details: Document Created

Revision: B
Revision Created: XX/XX/XXXX
Approved By: Patrick Powell
Revision Details:

1. Changed...
-

Revision: C
Revision Created: XX/XX/XXXX
Approved By: Patrick Powell
Revision Details:

1. Changed...